ADDING A NEW DRUG INTO CONSIS CREATE → Select CREATE PRODUCT 1 → Select the PRODUCT CODE field **STOCKCARD** → Scan the barcode of the new product to be added IN THE → Ensure the following fields are complete for the new drug: **CONSIS** Product Name, Product length (mm), Dosage Form, Package Unit SOFTWARE * Tick the EXCESS STOCK field if the item will typically have excess stored in the pharmacy e.g. those items ordered in bulk quantities HINT - You may wish to proceed directly to step 3 'Add channel allocation to new drug' by selecting ADD CHANNEL or select OK to save Choose an appropriate channel from the list of free channels (ensuring correct 2 PRFPARF package alignment). The free channel list may be filtered according to height by AND CHOOSE ticking or unticking the channel colour boxes; Red, Green, Blue or Yellow. AN → Adjust dividers / bottle channels to ensure the new drug fits snugly and is **APPROPRIATE** centred over the pick gap. CHANNEL 3 ADD → Select CREATE PRODUCT → Look up drug by either scanning (select **PRODUCT CODE** field) or typing CHANNEL (select PRODUCT SEARCH field) ALL OCATION → Select **EDIT** to enter the product stock card TO NEW Select ADD CHANNEL DRUG Select **OK** to save 5 → Select STOCK PACKAGES FILL DRUG → Select DELIVERY or EXCESS INTO → Select **OK** CHANNEL \rightarrow Scan product → Select Channel → Enter quantity filled into channel → Select **OK** 6 ADD NEW DRUG →Remember to also remove the old drug if it is no longer in the CONSIS TO DISPENSE (check for multiple channels) **SOFTWARE** ROBOTIC LIST TEST DELIVERY THROUGH DISPENSE SOFTWARE 7

If you are having trouble with your CONSIS PC please contact the Service line
1800 94 55 22 | 8am -8pm EST



HOW TO REMOVE / DELETE A DRUG FROM CONSIS


STEP 1.

Select Create Product

STEP 2.


Look up drug using one of the following options:

- Scan the barcode into the Product Code field
- Type the name into the **Product Name** field and select the correct option

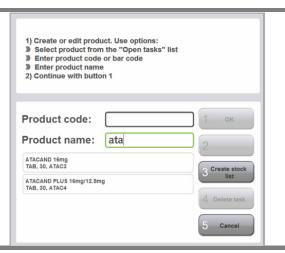

STEP 3.


- → Select **Edit** to open the stockcard
- → Select Delete Allocation

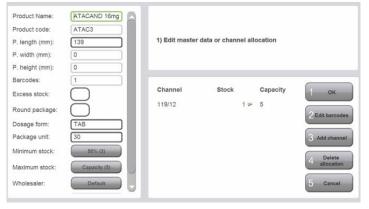

STEP 4.

There will be 2 options available:

- Delete this allocation
- Delete all allocations

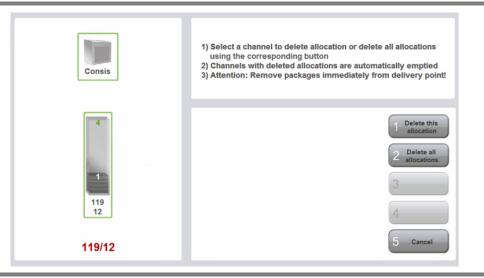



Extra Notes:

To REMOVE and UNASSIGN a drug from a channel:


• To remove and unassign a drug from only <u>one channel</u>, ensure that the correct channel is highlighted on the left side of the screen

→ Select Delete this allocation

- The CONSIS will empty the channel and ask you to confirm that it is empty
- To remove a drug with multiple channels from all assigned channels
- Select Delete all allocations
- The CONSIS will empty all allocated channels and ask you to confirm that they are empty

To REMOVE and DELETE a drug from CONSIS, the drug must first be unassigned from all allocated channels.

- Follow the steps to remove and unassign as above
- When you have unassigned all drug channels, option 4 on the right-hand side of the screen will now show Delete product, instead of Delete allocation
- Select Delete Product
- The Drug stock card and all information contained within it will now be deleted from CONSIS Navigator.

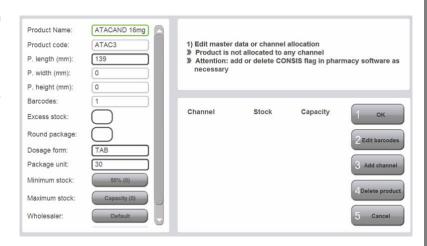

REMINDER: Important extra step

• Deleted drugs will also need to be removed from the Robotic Drug List in the Dispensing Software

If you are having trouble with your CONSIS PC please contact the Service line 1800 94 55 22 | 8am - 8pm EST

