CHANGING DRUGS IN CONSIS®		
	<u>TASK</u>	<u>PROCESS</u>
1	REMOVE OLD DRUG FROM CHANNEL	→ Select <b>DRUG MAINTENANCE</b>
		→ Select <b>DRUG INVENTORY</b>
		→ BROWSE for drug and then select the correct channel
		→ Select REMOVE DRUG FROM CHANNEL
2	PREPARE NEW CHANNEL	→ Adjust dividers / bottle channels to ensure the new drug fits snugly and is centred over the pick gap.
3	ADD NEW DRUG INTO CONSIS SOFTWARE (non-existing drug)	→ Select <b>DRUG MAINTENANCE</b>
		→ Select <b>ADD/CHANGE DRUG</b>
		→ Scan the new package barcode
		→ Using the CONSIS measuring tool, enter data on new drug: e.g. Name, length, channel colour and wholesaler details as applicable
4	ASSIGN CHANNEL TO NEW DRUG  (This can be done at stage 3 also)	→ Select <b>DRUG MAINTENANCE</b>
		→ Scan the new package barcode
		→ Select <b>ASSIGN CHANNEL</b>
		→ Select the new channel
5	FILL DRUG INTO CHANNEL	→ Select <b>FILL CONSIS</b>
		→ Scan the new package barcode and select channel
		<ul> <li>→ Scan and fill packages from rear of CONSIS</li> </ul>
6	ADD NEW DRUG TO DISPENSE SOFTWARE ROBOTIC LIST	→ Remember to also remove the old drug if it is no longer in the CONSIS (check for multiple channels)
7	TEST DELIVERY THROUGH DISPENSE SOFTWARE	
If you are having trouble with your CONSIS PC please contact the Service line 1800 94 55 22   8am - 8pm EST		