

## STARTING CON SIS SOFTWARE

### STEP 1 – Make sure the PC is ON.

- Check that the CON SIS PC is running- if not, turn on or restart computer. Make sure the screen is on.
- **The PC is usually located at the rear of the CON SIS, beneath the OPS flaps and shelf.**


### STEP 2 - Log into the CON SIS PC.

USERNAME:  
consisserver OR  
consisuser

PASSWORD:  
Consult the pharmacy  
manager



Picture of PC.

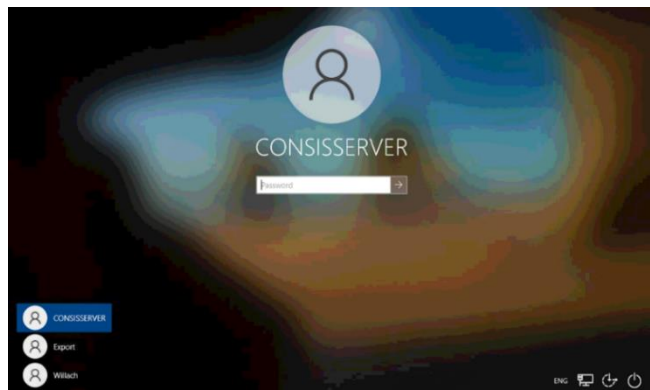

CON SIS computer login.

### STEP 3- Launch the software.

- Double click **Start CON SIS** (Willach icon) located on the desktop.
- The CON SIS server and Navigator software will launch.




Start CON SIS.



Picture of software launching.

### STEP 4- Minimise the CON SIS server screen.

- **The CON SIS Server screen must not be closed.** This is how the dispense software communicates with the robot.
- CON SIS Navigator is now ready to use

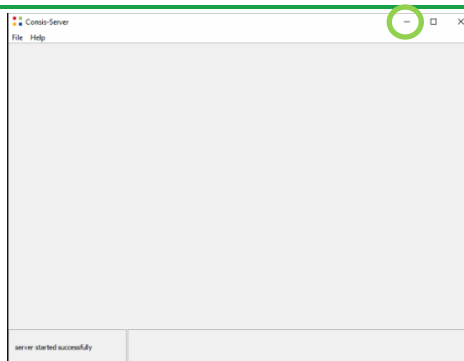

Picture of CON SIS server screen.



Picture of CON SIS Navigator screen.

### Hint: Multiple CON SIS server screens will interfere with each other.

- There should only be **two** CON SIS applications on the desktop taskbar: The **CON SIS Server** screen and the **CON SIS Navigator** software.
- **If there are more than two icons, close all CON SIS applications and restart (refer to Restarting CON SIS software).**



Picture of CON SIS PC taskbar with correct number of Willach icons.