

PERFORMING A STOCKTAKE

What is the stock take tool?

- The stock take tool generates a list of all CONSIS drugs. It shows the stock count, drug code and barcode for each product.
- The medications can be sorted in two ways:
 - Stocktake report- drugs are listed alphabetically by drug name. It shows the total stock count of each drug (i.e. if a drug has multiple channels, the quantities in all channel are combined). This report is used if stock counts are accurate and only need to be loaded/scanned into pharmacy records.
 - Stocktake by Channel- drugs are listed in order of channel location. The stock count for each channel is shown. This report is used if a count of each channel is required.

CONSI	S Report Tool			- 🗆 ×
Stock	Drug (Code	Product Name	Barcode
3	ABIS8		ABISART 150	9323610013507
2	ABIS9		ABISART 300	9323610013514
7	ABIS7		ABISART 75	9323610013491
3	ABIS12		ABISART HCTZ	9323610013545
3	ABIS11		ABISART HCTZ 300/12.5	9323610013538
5	ACIM6		ACIMAX	
Chann	Print S Report Tool el Stock	Save t	o CSV	Close – • × Barcode
101/2	3	METE6	METEX XR	9350299001143
101/3	4	PREX7	PREXUM 5	9316731000112
101/5	2	ME88	METHOBLASTIN	9315430051760
101/6	2	SYMB9	SYMBICORT	9322882010566
101/8	1	DIA8	DIAFORMIN	9323610000705
101/9	6	MELO8	MOXICAM	
(Drint	Save t	o CSV	Close



- Minimise Navigator.
- Find the Willach stock take tool on the
- desktop. Double click to launch.



Top: Picture of Stock take tool.

Right: Picture of the CONSIS PC desktop.

STEP 2. Choose the preferred report.

- Stocktake report OR
- Stacktaka by Channal

CONSIS Report Tool	Ē	-	×
	Stocktake Report		
	Stocktake by Channel		
	Configuration		

STEP 3.

Print report or export as an excel.

CONSIS Help Desk:

 \bigcirc

1800 94 55 22

2 2



STEP 4. Stocktake by Channel

- Park the machine and open the first shelf to check stock counts in each channel.
- If the stock count is different, write down the actual quantity on the report (to be entered into Navigator).

STEP 5. Enter	quantities	into	Navigator.
---------------	------------	------	------------

- Select the Reports tab> View all Shelves.
- Select a channel that requires stock adjustment.
- Type in the new/actual quantity and press OK/enter.
 - Navigator will move to the next channel.
 - Press OK/enter to move through the whole machine.
- After all quantities have been entered, generate a new Stocktake report to upload into pharmacy records.

			2) Specify stock and confirm			
Channel	Stock	Capacity	If actual stock is unknown, press button 2 and restock			
101/01	0	0	packages			
101/02	3	6				
101/03	4	12				
101/04	0	0	METEX XR ER-, 500mg, METE6			
101/05	2	8				
101/06	2	7	Stock (new):			
101/07	0	0	Create shelf			
101/08	1	5	121			
101/09	6	7	4.			
101/10	4	5	5 Back			
101/11	1	6				